

Installing Support Arms onto your CLEAN SHIELD™ seat

Support Arms can be installed when the seat is new or after it has been installed and used



Q: Are the Caps installed on the seat?

NO, Caps are not installed:

- > Follow Support Arm Installation

YES, Caps are installed:

- > First, follow Remove Caps Instructions
- > Second, follow Support Arms Installation

SUPPORT ARMS INSTALLATION

1
Confirm the Support Arms are on the correct side

2
Slide Support Arm into the mounting hole

3
Gently press "Lock Button"

4
Continue to slide into the mounting hole until the Lock Button "clicks" and is locked into place

REMOVE CAPS INSTRUCTIONS

A) Using a thin flat edge (putty or butter knife) pry the ring bumpers off the seat

B) Depress the "Lock Button" to unlock the Cap

C) Pull the Cap to remove it